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Instructions for use

Important information

1. Read these instructions for use before using for the first time.
2. Keep these instructions for use in a safe place.
3. Keep the INQUA[®] nasal rinse out of the reach of children.
4. Do not use the INQUA[®] nasal rinse for more than 2 months after opening the package. Mark the label on the bottle with the date it was opened.
5. After the expiry date, dispose of the INQUA[®] nasal rinse (see base of bottle).
6. Do not allow children to use the INQUA[®] nasal rinse without supervision.
7. If rinse solution gets into the eyes, flush the eyes with water.
8. The INQUA[®] nasal rinse is not to be ingested.
9. Use the INQUA[®] nasal rinse only in diluted form.
10. Only use the INQUA[®] nasal rinse together with the INQUA[®] nasal douche.
11. Follow the instructions for use for the INQUA[®] nasal douche as well.

Contents:	Natural saline solution	99.4 %
	Magnesium chloride hexahydrate	0.5 %
	Zinc acetate dihydrate	0.1 %
	Free of preservatives.	

Contents: 250 ml concentrated solution for 30 rinses

Areas of use

The INQUA[®] nasal rinse is used in diluted, isotonic form for thoroughly rinsing and cleaning the nasal mucous membrane. The nasal rinse assists in removing germs, dust and contaminants from the nose. This is particularly useful in dusty atmospheres or when pollen counts are high if the user suffers from hay fever.

Nasal rinses are also used in dry atmospheres – e.g., in heated or air conditioned rooms – to moisten the nasal mucous membrane. They can be used to loosen secretions in stuffy noses. A nasal rinse also helps in the treatment of inflamed paranasal sinuses and cold prevention.

Limitations, cross-reactions, side effects

Limitations

The INQUA[®] nasal rinse must not be used in cases of

- injury to the nasal roof, the paranasal sinus walls or the base of the skull
- palatine paralysis
- nosebleeds
- a predisposition to choking
- individual intolerance or hypersensitivity towards any of the ingredients in the saline solution listed above

Side effects

Correct use may cause mild irritation, such as a ticklish or stinging sensation in the nose, and an individual reaction (headache and more abundant mucus production). If you are using topical cold medicines (e.g., decongestant nose drops) at the same time, allow a reasonable pause between the two types of preparation.

No other limitations or cross-reactions with other substances have been reported with correct use.

Application

Note:

Only use the INQUA[®] nasal rinse together with the INQUA[®] nasal douche. The douche was developed specifically for use with the INQUA[®] nasal rinse.

- Unscrew the cap on the INQUA[®] nasal rinse supply bottle, but do not remove it from the bottle.
- Hold the bottle upright.
- Gently squeeze the bottle.
The concentrated saline solution will rise into the upper dosing chamber.
- Keep squeezing until the liquid reaches the dosage line.
- Take the cap off the supply bottle.
- For reasons of hygiene, avoid contact with the inside of the dosing device.
- Pour the dosed unit into the clean, prepared INQUA[®] nasal douche.
- Close the supply bottle securely with the screw cap.
- Fill the INQUA[®] nasal douche up to the bottom of the screw thread.
Use drinking water at a temperature of 37°C, i.e. normal body temperature.

Caution:

- Use only drinking water to prepare the nasal rinse solution. People with an elevated risk of infection (e.g., those with a suppressed immune system, people suffering from mucoviscidosis/CF) must use boiled or sterile water.
- Do not attempt to keep unused residue of the rinse solution. Dispose of all unused solution. Otherwise bacterial contamination may occur.
Rinse solutions must always be prepared fresh.

The rest of the procedure is described in the instructions for use of the INQUA[®] nasal douche.

Frequency of use

You can rinse the nose as needed up to 3 times a day with 250 ml isotonic saline solution. The recommended frequency is once per day. For rinsing and cleaning the nasal passages over a prolonged period, we recommend rinsing 2 - 3 times per week.

If your symptoms have not improved after 14 days, consult your doctor or chemist.

Storage

Always keep the bottle closed tightly.

Store out of direct sunlight, protect from excessive heat and freezing temperatures.

Legend

- Caution, see instructions for use
- Warning

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Restrictions, contre-indications, effets secondaires

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INQUA GmbH